



EasyOne Pro import Report Layout

Import

1. Utilities → Configuration → Report

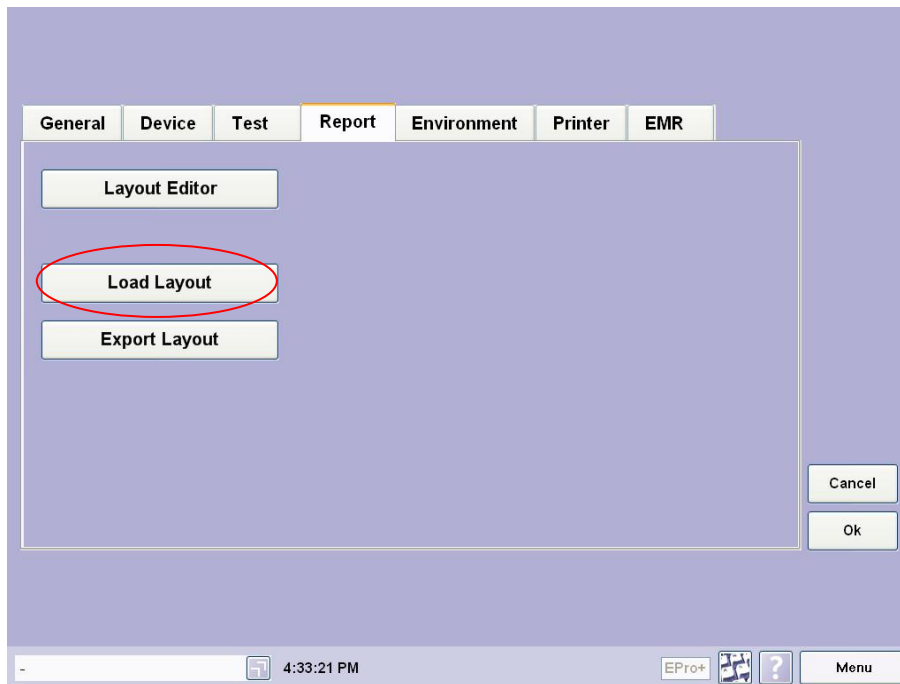


2. Browse the Report Layout

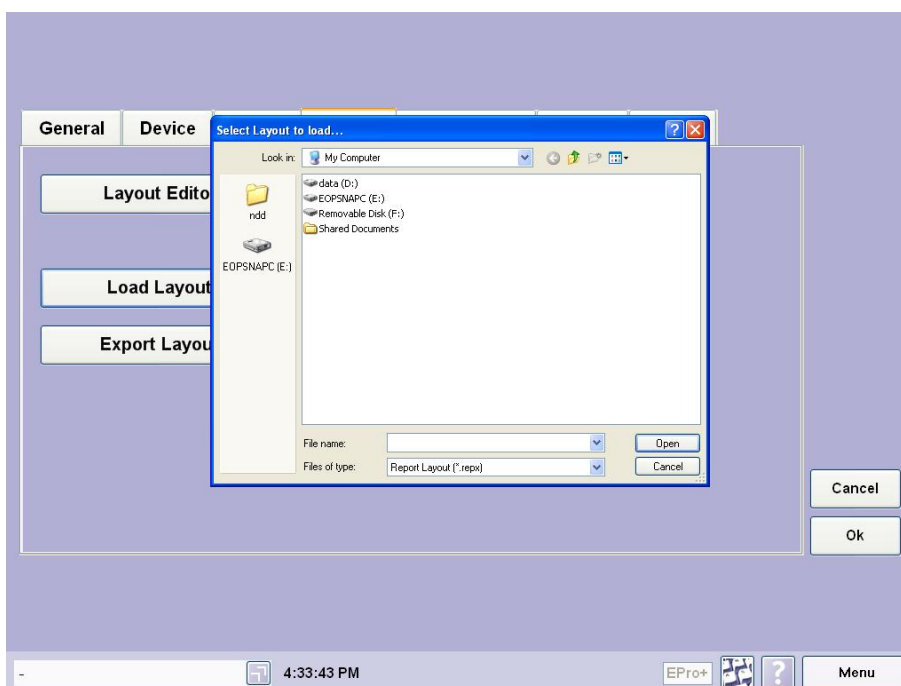


Select Layout

1. Select Patient
2. Press: Patient History

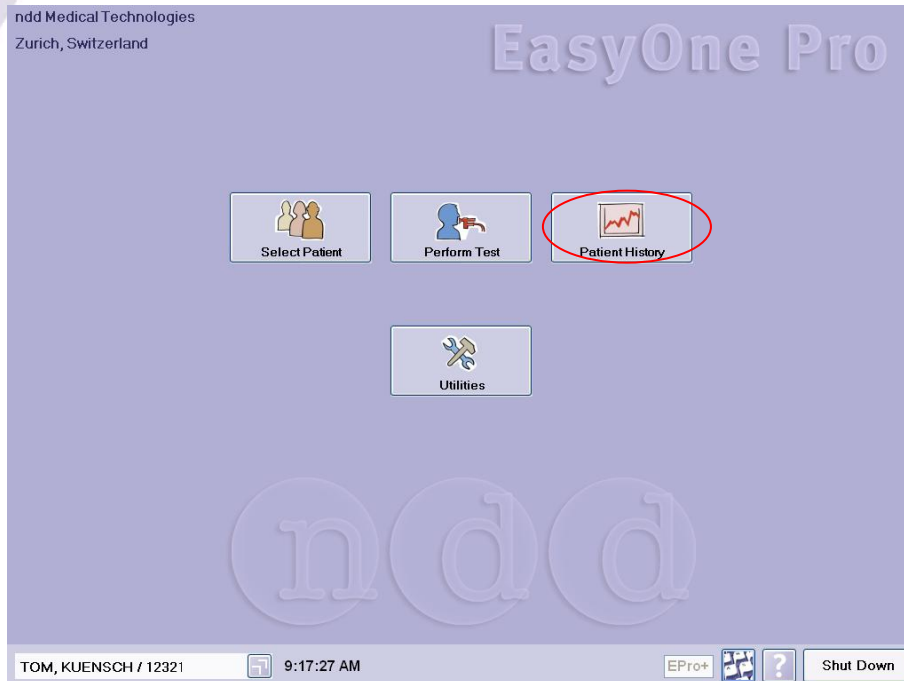

3. Select a Test
4. Press: Print Preview

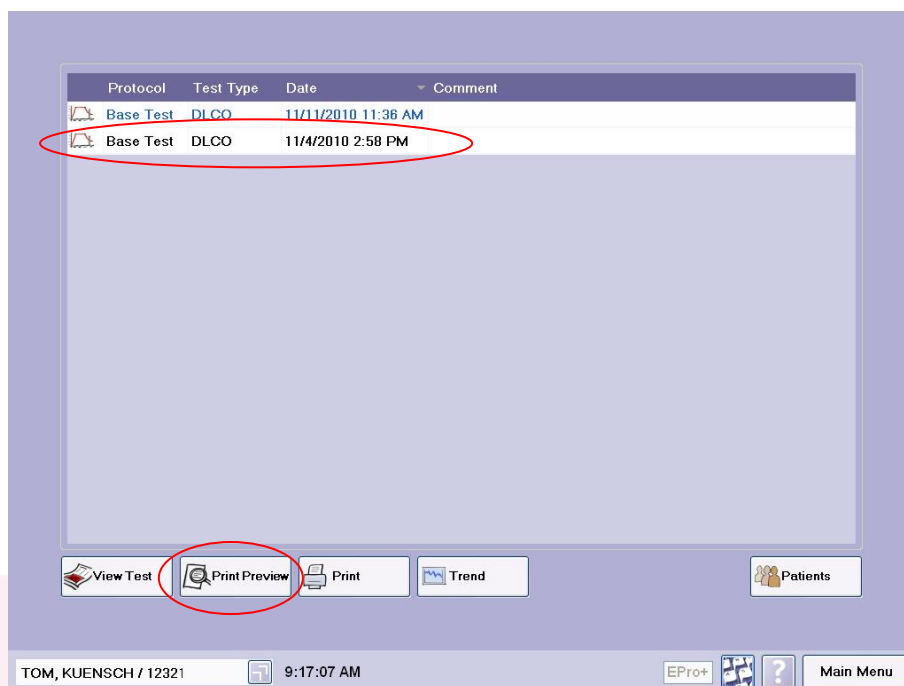

5. Select Layout

The screenshot shows a software interface for a medical device. At the top, there is a 'Layout Selection' window with a dropdown menu currently set to 'DLCO'. Below this, the main display area shows a report for a patient named 'TOM, KUENSCH'. The report is divided into several sections:

- Patient Information:** Name: TOM, KUENSCH; ID: 12321; Age: 67 (173/74); Weight: 180 cm; Height: 180 cm; Gender: male; Ethnic: Caucasian; Address: ...; Doctor: ...
- Test Information:** Test Type: DLCO; Test Date: 11/20/12 10:58:45 AM; Test Time: ...; Measurement: ...; Protocol: ...; Start Value: ...; End Value: ...
- Test Result:** A table with columns: Parameter, Unit, Value, Range, and Status.

| Parameter | Unit | Value | Range | Status |
|-------------------|------------|-------|-------------|--------|
| DLCO (ml/min/mmq) | ml/min/mmq | 24.4 | 21.0 - 27.0 | OK |
| DLCO (ml/min) | ml/min | - | 6.88 - 6.88 | - |
| DLCO (ml/min) | ml/min | - | 6.90 - 6.90 | - |
| DLCO (ml/min) | ml/min | 7.30 | 6.78 - 7.04 | OK |
| DLCO (ml/min) | ml/min | - | 3.60 - 3.60 | - |
| DLCO (ml/min) | ml/min | - | 3.5 - 3.5 | - |
- Quality:** DLCO (ml/min - ml/min/mmq) is marked as 'OK'.
- Graphs:** Two line graphs are shown, one for 'DLCO (ml/min/mmq)' and one for 'DLCO (ml/min)', both showing a sharp peak and then a gradual decline.

At the bottom of the interface, there is a status bar showing 'Page 1 of 1', '100%' zoom, and a footer with 'TOM, KUENSCH / 12321', '9:12:22 AM', 'EPro+', and a 'Return' button.