

## EasyOne Pro Database Transfer

### 1 Introduction


Electronically transferring the EasyOne Pro database from a customer to ndd can cause several problems: The database can be too large for email transmission; the database may be blocked by Virus filters; or the transmission may not be secure enough.

This Application Note describes how the EasyOne Pro database can be securely transferred to an FTP server owned by ndd.

### 2 Step 1: Export Patient Database from EasyOne Pro

The exported patient database is encrypted.


1. Plug in a USB Memory Stick to the EasyOne Pro
2. Go to Utilities / Advanced / pw: 8005 and press Login/ Export
3. Select Patient Data

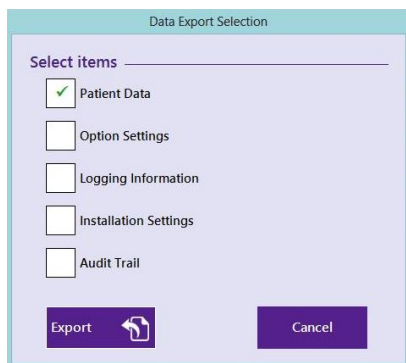

4. Press Export
5. In the window, browse to the USB Memory Stick (Removable Disk) and select the top folder
6. Press OK
7. Remove the USB Memory Stick from the EasyOne Pro

### 3 Step 2: Transfer the database to ndd

1. Connect the USB Memory Stick to your computer
2. Request a file share link from [support@nnd.ch](mailto:support@nnd.ch) or use the general file upload link below if you do **not** upload any sensitive data
3. Copy the link to your Browser (Internet Explorer, Microsoft Edge, Google Chrome, Firefox, ...) and press Enter  
General upload link for no sensitive data: <https://nndmedtech.sharepoint.com/:f:/s/filetransfer/Eslcc8qp6ZhPo9mxun6fPKUBAIHgl-Ruun8fEgRpQJowQUg?e=vz2Tvs>
4. An upload folder of ndd's File Transfer SharePoint site will be opened
5. Drag and drop the copied patient database "EasyWarePro.sqlite" to the folder
6. You might not see the file directly because it's still uploading

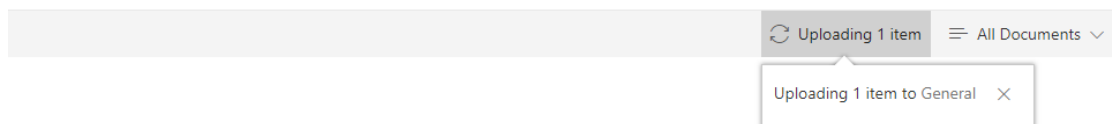

7. Inform ndd by Email or phone that you have uploaded the database  
Phone: +41 44 445 29 70  
Email: [support@nnd.ch](mailto:support@nnd.ch)

### 4 Alternative Transfer

Alternatively you might also use an internet service such as Dropbox: <http://www.dropbox.com/> to transfer the data.