

Author: JSc  
Date: 23.12.2021  
Confidentiality: public

## Recommended Printers for EasyOne Pro/LAB

### 1. EasyOne Pro/LAB V05 (SN 55x'xxx & SN 65x'xxx)

Printers can be connected directly via USB or shared and accessed over the network.

Printing options:

- PCL standard
- direct to printer
- network printer
- print to PDF



Choose the printer model based on the compatibility of the operating system.

**Requirement:** Compatibility with Windows 8 embedded.

### 2. EasyOne Pro/LAB V04 (SN 50x'xxx & SN 60x'xxx)

This chapter lists the recommended printers for EasyOne Pro/LAB V04.

#### 2.1 Pre-installed Printers on EasyOne Pro/LAB

The pre-installed printers listed below do not require any installation on the EasyOne Pro/LAB:

- HP Universal Printer PCL5 (supports most network printers)
- HP OfficeJet H470 (on all EasyOne Pro/LAB disk images)
- HP DeskJet 6940 (on all EasyOne Pro/LAB disk images except V03.02 ndd A)
- HP OfficeJet 6100 (on EasyOne Pro/LAB disk image V03.02 ndd F and higher)
- HP OfficeJet Pro 8100 (on EasyOne Pro/LAB disk image V03.02 ndd F and higher)



**Note:** The EasyOne Pro/LAB software image version can be determined via [Utilities > Configuration > About > Details](#)

## 2.2 Recommended Printers (Tested)

The printer drivers listed below can be downloaded from the NDD Website

[Reports & Printers](#) | [Resources](#) | [NDD Medical](#)

- Epson Series N11, NX120, S22, SX125, T12, T13, T22, T22E, TX120, TX121
- Epson WorkForce WF-2530
- HP DeskJet 3000
- HP Inkjet 1000
- HP OfficeJet 6000

For instructions on how to install a printer driver on an EasyOne Pro/LAB V04, please refer to our Application Note [Using Printers with EasyOne Pro](#)

## 2.3 Printers (Untested; but Printer Drivers Available)

HP			
Business Inkjet 1200	DeskJet 6980	LaserJet M1005	OfficeJet 150
DeskJet 1000	DeskJet 960	LaserJet P1005	OfficeJet 6000
DeskJet 1015	DeskJet F2400	LaserJet P1100, P1560, P1600 XPS	OfficeJet 6100
DeskJet 2515	InkJet 1000	LaserJet P2014	OfficeJet J4580
DeskJet 3000	LaserJet 1018	LaserJet P2035	OfficeJet J4660
DeskJet 460	LaserJet 1020, 1022 PLUS	LaserJet P2055	OfficeJet Pro 8000
DeskJet 5440	LaserJet 1320	LaserJet P3015	OfficeJet Pro 8100
DeskJet 5940	LaserJet 1566	LaserJet Pro 300/400 Color M351/M451	Series D1400, 2400 ,4200
DeskJet 6520	LaserJet 4700	OfficeJet 100	Series D5500

Brother'	Canon	Epson	Kyocera
HL-2140/2150N /2170W	i-SENSYS LBP6020	InkJet L100, L200	FS-1030D
HL-2250DN	PIXMA iP1800	Series T10, T11	Universal Driver
HL-5340D	PIXMA iP2700	Stylus C79	
MFC-7860DW	PIXMA iP4900	XP-312	
MFC-J435W	PIXMA iP6220D		
	PIXMA iP90		
	PIXMA MG3570		
	PIXMA MX452		
	PIXMA MX527		
	PIXMA MX850		

Lanier	Lexmark	Samsung
LP020C	C543	ML-1210
	Universal Driver	ML-1750
		SCX-4200
		Universal Driver